## Warnings and Primary Remarks

- ⚠ Use of NanoEx must be by, or on the order of a physician (or properly licensed practitioner).
- ⚠ NanoEx is a sterile and single-use product and should not be re-used.  
Re-use may lead to increased risk of infection or cross contamination.
- ⚠ Do not use the product if there is evidence of packaging damage or after the expiration date.
- ⚠ NanoEx should be held at room temperature.
- ⚠ Keep the product away from sunlight or excessive moisture.
- ⚠ If you experience any allergic reactions, discontinue use of the product, immediately contact your healthcare professional.
- ⚠ Please refer to the directions for use and carefully and precisely follow them.

## Indications (Intended Purpose):

Management of:

- Burn wounds
- Diabetic foot ulcers
- Pressure ulcers
- Traumatic ulcers
- Skin grafts
- Donor sites
- Surgical wounds
- Cavity wounds
- Infected and non-infected wounds
- Wounds with minimal to moderate exudate

## Contraindications

- Malignancy in the wound
- Unexplored fistula
- Untreated osteomyelitis
- Necrotic tissue with eschar present
- Exposed blood vessels or organs (heavy bleeding)
- Allergic reaction

## Directions for Use

- 1) Check the Indication. NanoEx is indicated for wounds/ulcers with minimal to moderate exudate. Consider using an absorbant secondary dressing if the wound exudate is high.
- 2) Wash your hands, use surgical gloves, work in sterile setup and preparations.
- 3) Wash the wound/ulcer and the area around it (> 1cm margin) and remove the contaminants using standard solutions (normal saline). Please do not use irritative or cytotoxic cleansing solutions.
- 4) If debridement of necrotic or hyperkeratotic tissue is required, let a physician (or properly licensed practitioner) do that.
- 5) Gently dry the skin around the wound. Take precautions not to cause any bleeding.
- 6) Measure the size of the wound/ulcer. Open the package. Cut the NanoEx dressing and consider covering 1cm around it as a safe margin for complete adhesion and protection.
- 7) Put the dressing on the wound/ulcer. Then moisten it drop by drop with appropriate amount of normal saline or sterile distilled water, in such a way that no extra water is left.
- 8) If the dressing is not completely adhered to the wound/ulcer surface, gently press on those areas with sterile gauze pads or medical swabs until it is completely adhered to the wound surface ensuring no air is trapped between the wound surface and NanoEx.

## Side Effects

In the case of irritation, discontinue use of product.

## Intended Users (Target Users)

Health care professionals and technicians in the field of wound care and healing

## Intended Environment and Place of Use

This product has no restriction or limitation for place of use.

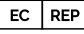
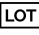
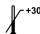
## Target Population

Adults (excluding pregnant)

## Target Location (Application Site in the Body)

In direct contact with patient's breached skin

## Symbols

	Manufacturer		Do not re-sterilize
	Authorized representative in the European Community		Do not re-use
	Date of manufacture		Consult instructions for use
	Use-by date		Do not use if package is damaged
	Lot Code		Keep dry
	Catalogue number		Keep away from sunlight
	Sterilized using irradiation		Latex free
			Upper limit of temperature

 AKTISCHICHT Nano F. N. Teb ACTA1 holding, No. 1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, IRI

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