Application

- 1: Sterilize your hands before use with detergent and water or sanitizer.
- 2: Dry your hands and put on sterile gloves
- 3: Rinse the wound in washing solution (for example normal saline) to loosen and remove surface contaminations on the wound.
- 4: Wet a soft piece of cloth or towel, while gently applying mild soap to clean surrounding areas of the wound. Do not allow soap to touch the wound.
- 5: Use sterile tweezers to remove any dirt or debris that still appears after washing.
- 6: Gently dry the skin around the wound taking precaution not to cause any bleeding.

How to use the dressing

- Firstly, the required amount of powder should be considered according to the size of the wound.
- Then open the package and apply the powder to the wound.
- NanoExpo dressing can be used for 3 to 10 days, depending on the type of wound and the diagnosis of the specialist. It is recommended to change the dressing from 48 to 72 hours. However, the time of applying new NanoExpo will depend on the condition of the patient as well as the degree of the wound healing.
- Use an appropriate wound cleaner (normal saline) to cleanse the wound site prior to applying a new NanoExpo.

Indications (intended purpose):

Management of:

- Cavity wounds (Tunneling)
- Burn wounds
- Diabetic foot ulcers
- Pressure ulcers
- Traumatic ulcers
- Skin grafts
- Donor sites
- Surgical wounds
- Wounds with moderate to heavy exudate

Contraindications

Malignancy in the wound

- Unexplored fistula
- Untreated osteomyelitis
- Necrotic tissue with eschar present
- Exposed blood vessels or organs (heavy bleeding)
- Allergic reaction

Side effects

In the case of irritation, discontinue use of product.

Intended Users (target users)

Health care professionals and technicians in the field of wound care and healing

Intended environment and place of use

This product has no restriction or limitation for place of use.

Target population

Adults (excluding pregnant)

(It can be used for both male and female)

Target location (application site in the body)

In direct contact with patients breached skin

Warnings

- ⚠ Do not use if package is damaged.
- ⚠ Store in a dry and cool place preferably at room temperature.
- ⚠ Keep away from sunlight or excessive moisture.
- ⚠ If you experience any allergic reactions, discontinue use of the product, immediately contact your healthcare professional.
- ⚠ NanoExpo is for single-use only and should not be re-used.
Re-use may lead to increased risk of infection or cross contamination.

Please contact Nano Narin Teb patient support for further information and directions.

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