AntiScar Gel

Warnings:

- ⚠ Do not use around the eyes, on mucosal membranes, third-degree burns, or open wounds.
- ⚠ The product should not be used on dermatological conditions with broken skin.
- ⚠ If the gel is not completely dry, it may stain clothing.
- ⚠ If you experience any allergic reactions, discontinue use of the product and contact your healthcare professional.
- ⚠ For external use only.
- ⚠ This product does not require special disposal methods. Please follow your local disposal regulations.
- ⚠ This product meets the requirements of ISO10993.
- ⚠ Do not use the product if there is evidence of packaging damage or after the expiration date.
- ⚠ Keep away from direct sunlight.
- ⚠ Please refer to the directions for use and carefully and precisely follow them.

Indications (Intended Purpose):

Management of old and new keloid and hypertrophic scars resulting from burn, surgical procedure, wounds and trauma.

Drug interaction

There is no known and reported drug interaction.

Contraindications

It is contraindicated in patients with known hypersensitivity to silicone or any of the listed ingredients.

Directions for Use

- 1.First, clean and dry the area
- 2.Apply a thin layer of gel to the scar and massage gently in order to achieve better penetration.
- 3.The gel should be used twice a day

Treatment may take several weeks, depending on the size of existing scars.

Recommended duration of treatment

It is recommended as a minimum treatment of 60–90 days (twice a day). Larger and older scars may take longer. Continued use is recommended until no further improvement is seen.

Storage condition

Store in a dry place at room temperature.
Protect from direct sunlight and moisture.
Keep it out of the reach of children.
Do not freeze.

Intended Users (Target Users)

This product does not need to be applied by a healthcare professional, and the patient can use it independently without the help of a specialist.
NTScar should not be applied over other skin treatments without the advice of your physician.

Target Population

Children and adults

Size and catalogue number:

5 ml	NS/01/01
10 ml	NS/02/01
15 ml	NS/03/01

Symbols

Manufacturer

Date of manufacture

Use-by date

Lot Code

Catalogue number

Consult instructions for use

Do not use if package is damaged

Keep dry

Keep away from sunlight

Avoid freezing

Manufactured by: Nano Fanavaran Narin Teb

No.1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, Iran

Please report any adverse events related to this product to the email address of Nanonarin Teb Company (info@nanonarin.com) or to the General Department of Medical Equipment of Iran.

Please contact Nano Narin Teb patient support for further information and directions.

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