Wound Antiseptic and Antibacterial Solution

Size

30ml
60ml
100ml

Warnings:

- ⚠ Only for external use on skin and wounds.
- ⚠ Do not use for infusion or injection.
- ⚠ Do not use if package is damaged.
- ⚠ Do not ingest.
- ⚠ Avoid contact with eyes
- ⚠ Do not contaminate the spray nozzle.
- ⚠ This product is non-sterile and should not be re-used.
- ⚠ If you experience any allergic reactions, discontinue the use of this product, immediately contact your physician.
- ⚠ The maximum time to change the product is one hour and the maximum duration of use is less than 24 hours.
- ⚠ Please refer to the instructions for use below and read it carefully.
- ⚠ Contact us in case of any complications.
- ⚠ Dispose the product safely.

Indications (intended purpose):

For cleansing, decontamination and infection treatment of:

- Diabetic foot ulcers
- Pressure ulcers (such as bedsores)
- Surgical wounds
- Traumatic wounds
- Burns

Contraindications:

- Do not use if there is a history of allergy to any of the ingredients.
- Do not use on the central nervous system or meninges, middle or inner ear, eyes or cartilage.
- Mixing the Wound Antiseptic and Antibacterial Solution with other wound cleansing soaps, lotions, ointments, oils, or enzymes may lower effectiveness

Directions for Use

1. Skin Cleansing

- Spray the solution on the desired area of the skin, or soak a sterile gauze in this solution and wipe the desired area.

2. Wound cleansing

- Spray the solution on the entire wound bed as well as the edges of the wound. Avoid flooding the wound with the solution.
- Or, wet the sterile gauze with the solution and then, clean the wound with it. Then cover the wound with a proper dressing.
- Do this once a day, make sure the wound bed remains moist until the dressing is changed.

Side Effects

In very rare cases there may be a mild burning and itching sensation after application which will go away after a short time.

Intended Users (target users)

Healthcare professionals and technicians in the field of wound care and healing

Target population

Adults (excluding pregnant)

Storage

- store at room temperature below 30°C and away from direct sunlight.
- Do not refrigerate or freeze.

sizes and catalogue numbers:

30ml	ED/01/01
60ml	ED/02/01
100ml	ED/03/01

Symbols

Manufacturer

Consult instructions for use

Date of manufacture

Keep away from sunlight

Use-by date

Non-sterile

Lot Code

Do not re-use

Catalogue number

Lower and upper limits of temperature

Manufactured by: Nano Fanavaran Narin Teb

No. 1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, Iran

Please report any adverse events related to this product to the email address of Nano Fanavaran Narin Teb (info@nanonarin.com) or the General Department of Medical Equipment of Iran.

Please contact Nano Narin Teb patient support for further information and directions.

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