Transparent Tape

Transparent Tape

Description

“Transparent tape” is made of medical polyurethane film, coated with acrylic adhesive, providing maximum versatility and is the affordable choice for skin protection and water proof cover dressing.

Directions for Use

1. When using, the area around the tape should be clean and dry.
2. Cut the appropriate piece out of the product with scissors. Its size should be sufficient to cover about 2.5 cm around the wound.
3. In the next step, gently affix one side of the tape to the skin. Then gently press to squeeze the air out of the transparent tape.
4. In the last step, the tape at the other end of the product is turn off. Then continue to gently press to squeeze out the air.
5. Removing the tape should be done gently and carefully, from the edges toward the center, to prevent the removal of the superficial layer of the skin's epidermis.
6. On average, this product needs to be changed every 3 to 5 days

Indications (Intended Purpose):

- Dressing fixation
- Catheter fixation
- Transparent film for vacuum therapy
- Preventing the primary dressing from getting wet
- This tape can protect skin from external friction and cross-infection of wounds
- Maintaining a moist environment to speed up wound healing and facilitate and promote autolytic debridement.

Contraindications

- Infected wounds
- Wounds with fragile periwound skin

Intended user:

Children and adults, infants and pregnant women

Transparent Tape

Warnings:

- ⚠ This tape is not intended for use as the primary dressing and should not be applied directly to the wound; it requires intact periwound skin for proper adhesion.
- ⚠ Make sure to apply to clean and dry skin for proper adhesion.
- ⚠ Stop any bleeding at the site before applying the tape.
- ⚠ Do not stretch the tape during application as tension can cause skin trauma.
- ⚠ If you experience skin irritation, discontinue use of the product, immediately contact your healthcare professional.
- ⚠ If you observe leakage, loosening, or curling of the edges of the tape, you should replace it.
- ⚠ May cause maceration.
- ⚠ Keep away from direct sunlight.
- ⚠ Contact us in case of complications.
- ⚠ This product is for single-use only and should not be re-used.
- ⚠ Disposal: This product should be disposed based on the hospital or healthcare professional advice. (According to local regulations and procedures)
- ⚠ Do not use the product if there is evidence of damage to the protective layers or if it has passed its expiration date.
- ⚠ Are not recommended for use on wounds with moderate to heavy exudate
- ⚠ Do not trap the tape too tight on your body part which may cause cutting or arising blood circulation slow down.

Storage condition

Store in a cool and dry place, below 25°C. Protect from direct sunlight and moisture.

size and catalogue number:

5cm × 10m	NT/01/01
10cm × 10m	NT/02/01
15cm × 10m	NT/03/01

Transparent Tape

Symbols

	Manufacturer		Upper limit of temperature
	Date of manufacture		Do not re-use
	Use-by date		Consult instructions for use
	Lot Code		Do not use if package is damaged
	Catalogue number		Keep dry
			Keep away from sunlight

Manufactured by: Nano Fanavaran Narin Teb

No. 1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, Iran

Please report any adverse events related to this product to the email address of Nanonarin Teb Company (info@nanonarin.com) or to the General Department of Medical Equipment of Iran.

Please contact Nano Narin Teb patient support for further information and directions.

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