Foam dressing

Models

- NanoFoam
- NanoFoam Silicone

Size

- 10 cm × 10 cm
- 15 cm × 15 cm
- 20 cm × 20 cm

Warnings:

- ⚠ Contact us in case of any complications.
- ⚠ Do not use if package is damaged.
- ⚠ Store in a dry and cool place preferably at room temperature.
- ⚠ Keep away from sunlight or excessive moisture.
- ⚠ If you experience any allergic reactions, discontinue use of the product, immediately contact your healthcare professional.
- ⚠ This product is for single-use only and should not be re-used.
- ⚠ Re-use may lead to increased risk of infection or cross contamination.
- ⚠ In the case of redness of skin, discontinue use of product.
- ⚠ Disposal: This product should be disposed based on the hospital or healthcare professional advice. (According to local regulations and procedures)

Indications (intended purpose):

- Diabetic foot ulcers
- Pressure ulcers
- Burn wounds
- Traumatic ulcers
- Skin grafts
- Donor sites
- Surgical wounds
- Vascular ulcers
- Wounds with moderate to heavy exudate

Contraindications:

- Necrotic tissue with eschar
- Dry wounds
- Allergic reaction

Directions for Use

1. Check the Indications. This product is indicated for wounds with **moderate to heavy exudate**.
2. Wash your hands, use surgical gloves, work in sterile set-up and preparations.
3. Wash the wound and the area around it (> 2.5 cm margin) and remove the contaminants using standard solutions (normal saline). Please do not use irritative or cytotoxic cleansing solutions.
4. If debridement of necrotic or hyperkeratotic tissue is required, let a physician (or properly licensed practitioner) do that.
5. Gently dry the skin around the wound. Take precautions not to cause any bleeding.
6. Measure the wound to select the appropriate dressing size and cover a distance of 2.5 cm (1 inch) of the surrounding skin.
7. When using NanoFoam Silicone, remove the first release liner and apply the adherent side to the wound. Remove the other release liner and smooth down on the skin.
8. For both NanoFoam and NanoFoam Silicone, use a bandage or tape to hold the dressing in place.
9. The maximum time to replace the product is **72 hours** and the maximum duration of use is less than 30 days.
10. There is no limit on the number of times the product can be used. However, the maximum duration of use is less than 30 days. In case of insufficient absorption of exudate and foam saturation, use another treatment method.

How to remove

1. When changing the dressing, carefully peel off the foam dressing from a corner.
2. Clean the wound and if needed, apply a new foam dressing.

Intended Users (target users)

Healthcare professionals and technicians in the field of wound care and healing

Target population

Adults (excluding pregnant)

Target location (application site in the body)

In direct contact with patients' breached skin

Intended environment and place of use

This product has no restriction or limitation for place of use.

sizes and catalogue numbers:

NanoFoam	10×10	NF/01/01	NanoFoam Silicone	10×10	NF/11/01
	15×15	NF/02/01		15×15	NF/12/01
	20×20	NF/03/01		20×20	NF/13/01

Symbols

Manufacturer

Date of manufacture

Use-by date

Lot Code

Catalogue number

Sterilized using ethylene oxide

Do not re-sterilize

Do not re-use

Consult instructions for use

Do not use if package is damaged

Keep dry

Keep away from sunlight

Manufactured by: Nano Fanavaran Narin Teb

No. 1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, Iran
Packaging and Sterilization, Made in China

Please report any adverse events related to this product to the email address of Nano Fanavaran Narin Teb (info@nanonarin.com) or the General Department of Medical Equipment of Iran.

Please contact Nano Narin Teb patient support for further information and directions.
Tel: +9821-91076061, www.nanonarin.com, info@nanonarin.com

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