Multi-functional healing gel

Size

- 15ml
- 30ml
- 50ml

Warnings:

- ⚠ Contact us in case of any complications.
- ⚠ Sterile if undamaged, unopened Pack.
- ⚠ Store in a dry and cool place preferably at room temperature.
- ⚠ Keep away from sunlight or excessive moisture.
- ⚠ If you experience any allergic reactions, discontinue use of the product, immediately contact your healthcare professional.
- ⚠ Disposal: This product should be disposed based on the hospital or healthcare professional advice. (According to local regulations and procedures)
- For external use only on skin wounds.
- ⚠ Do not use for infusion or injection.
- ⚠ Do not ingest.
- ⚠ Use with secondary dressing to prevent evaporation of water content.
- ⚠ The maximum time to replace the product is 72 hours and the maximum duration of use is less than 30 days.
- ⚠ Do not use the product if there is evidence of packaging damage or after the expiration date.
- ⚠ This product meets the requirements of ISO10993.
- ⚠ Healogel is a sterile and single-use product and should not be re-used.

Indications (intended purpose):

The Wound Gel is intended to cleanse and balance wound moisture by donating water and absorbing exudate for the management of:

- infected and uninfected wounds with low to moderate exudate
- Pressure ulcers
- First and second-degree burns
- Partial and full thickness wounds
- Surgical incisions
- Diabetic ulcers
- Donor sites and grafts
- vascular ulcers

Contraindications:

- Allergic reaction

Directions for Use

1. Wash the wound and the surrounding area. (Exidal can be used for this purpose.)
 2. Apply sufficient amount of hydrogel on wound. (Regarding to size and situation of wound)
 3. Apply second dressing like gauze.
- For infected wounds with high exudate, use an appropriate absorbent secondary dressing based on the level of wound exudate.

How to remove

1. The gel can be removed from the wound by rinsing with sterile normal saline solution.
2. the frequency of dressing changes depends on the clinical condition of the wound and the amount of exudate produced. Change the wound dressing as directed by a healthcare professional.

Intended Users (target users)

Healthcare professionals and technicians in the field of wound care and healing

Target population

Adults (excluding pregnant)

Target location (application site in the body)

In direct contact with patients’ breached skin

Intended environment and place of use

This product has no restriction or limitation for place of use.

Side Effects

In the case of irritation, discontinue use of product.

size and catalogue number:

15ml	30ml	50ml
HG/01/01	HG/02/01	HG/03/01

Symbols

Manufacturer

Date of manufacture

Use-by date

Lot Code

Catalogue number

Sterilized using radiation

Upper limit of temperature

Do not re-sterilize

Do not re-use

Consult instructions for use

Do not use if package is damaged

Keep dry

Keep away from sunlight

Manufactured by: Nano Fanavaran Narin Teb

No. 1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, Iran

Please report any adverse events related to this product to the email address of Nano Fanavaran Narin Teb (info@nanonarin.com) or the General Department of Medical Equipment of Iran.

Please contact Nano Narin Teb patient support for further information and directions.

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