Hydrocolloid dressing

Models

- NanoColloid
- NanoColloid Thin

Size

- 10 cm × 10 cm
- 15 cm × 15 cm
- 20 cm × 20 cm

Warnings:

- ⚠ Contact us in case of any complications.
- ⚠ Do not use if package is damaged.
- ⚠ Store in a dry and cool place preferably at room temperature.
- ⚠ Keep away from sunlight or excessive moisture.
- ⚠ If you experience any allergic reactions, discontinue use of the product, immediately contact your healthcare professional.
- ⚠ This product is for single-use only and should not be re-used.
- ⚠ Re-use may lead to increased risk of infection or cross contamination.
- ⚠ In the case of redness of skin, discontinue use of product.
- ⚠ Disposal: This product should be disposed based on the hospital or healthcare professional advice. (According to local regulations and procedures)

Indications (intended purpose):

- Traumatic ulcers
- Pressure ulcers
- Burn wounds (stage 1 and 2)
- Wounds with low to moderate exudate
- Diabetic foot ulcers
- Arterial and venous ulcers
- Surgical wounds

Contraindications:

- Infected wounds
- Wounds with exposed tendon or bone
- Wounds with heavy exudate
- Allergic reactions

Directions for Use

1. Check the Indication. This product is indicated for wounds with **low to moderate exudate**.
2. Wash your hands, use surgical gloves, work in sterile setup and preparations.
3. Wash the wound and the area around it (> 3cm margin) and remove the contaminants using standard solutions (normal saline). Please do not use irritative or cytotoxic cleansing solutions.
4. If debridement of necrotic or hyperkeratotic tissue is required, let a physician (or properly licensed practitioner) do that.
5. Gently dry the skin around the wound. Take precautions not to cause any bleeding.
6. Measure the wound to select the appropriate dressing size and cover a distance of 3.0 cm (1¼ inch) on the entire skin around the wound.
7. Hold NanoColloid dressing between hands to warm it.
8. **The side covered with the white release paper should be adhered to the wound.**
9. While peeling off the white paper, put the NanoColloid on the wound and gently apply pressure so that the dressing sticks to the wound.
10. Remove the transparent tape and stick the rest of the dressing to the wound.
11. Gently press the NanoColloid until it adheres completely.
12. NanoColloid is a primary dressing and should be in **direct contact with the wound bed**.
13. The maximum time to replace the product is **72 hours** and the maximum duration of use is less than 30 days.

How to remove

1. For removing the dressing, gently lift one corner of the dressing upwards.
2. Roll it away from the wound while keeping the skin away.

Intended Users (target users)

Healthcare professionals and technicians in the field of wound care and healing

Intended environment and place of use

This product has no restriction or limitation for place of use.

Target population

Adults (excluding pregnant)

Target location (application site in the body)

In direct contact with patients' breached skin

sizes and catalogue numbers:

NanoColloid	10×10	NC/01/01	NanoColloid Thin	10×10	NC/11/01
	15×15	NC/02/01		15×15	NC/12/01
	20×20	NC/03/01		20×20	NC/13/01

Symbols

	Manufacturer		Do not re-sterilize
	Date of manufacture		Do not re-use
	Use-by date		Consult instructions for use
	Lot Code		Do not use if package is damaged
	Catalogue number		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight

 **Manufactured by: Nano Fanavaran Narin Teb**
No. 1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, Iran
Packaging and Sterilization, Made in China

Please report any adverse events related to this product to the email address of Nanonarin Teb Company (info@nanonarin.com) or to the General Department of Medical Equipment of Iran.

Please contact Nano Narin Teb patient support for further information and directions.
Tel: +9821-91076061, www.nanonarin.com, info@nanonarin.com

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