Interactive Regenerating Wound Dressing Powder

Warnings:

- ⚠ Sterile if undamaged, unopened pack.
- ⚠ Contact us in case of complications.
- ⚠ Disposal: This product should be disposed based on the hospital or healthcare professional advice. (According to local regulations and procedures)
- ⚠ This product meets the requirements of ISO10993.
- ⚠ Do not use if package is damaged.
- ⚠ Store in a dry and cool place preferably at room temperature.
- ⚠ Keep away from sunlight or excessive moisture.
- ⚠ If you experience any allergic reactions, discontinue use of the product, immediately contact your healthcare professional.
- ⚠ This product is for single-use only and should not be re-used.
- ⚠ Re-use may lead to increased risk of infection or cross contamination.
- ⚠ In the case of redness of skin, discontinue use of product.
- ⚠ A suitable secondary dressing should be used on NanoExpo based on the clinician's diagnosis and the wound condition.

Indications (intended purpose):

Management of:

- Cavity wounds (Tunneling)
- Burn wounds
- Diabetic foot ulcers
- Pressure ulcers
- Traumatic ulcers
- Skin grafts
- Donor sites
- Surgical wounds
- Wounds with moderate to heavy exudate

Contraindications:

- Malignancy in the wound
- Unexplored fistula
- Untreated osteomyelitis
- Necrotic tissue with eschar present
- Exposed blood vessels or organs (heavy bleeding)
- Allergic reaction

Wound bed preparation

1. Check the indications before use.
2. Wash your hands, use surgical gloves, and work in sterile setup and preparations.
3. Wash the wound and the area around it (> 1cm margin) and remove the contaminants using standard solutions (normal saline). Do not use irritative or cytotoxic cleansing solutions.
4. If debridement of necrotic tissue or hyperkeratosis is required, let a physician (or properly licensed practitioner) do that.
5. Gently dry the skin around the wound taking precautions not to cause any bleeding.

How to use the dressing

1. Open the primary packaging from the tear notch.
2. Hold the NanoExpo glass packaging in your hand, peel off the hologram sticker, remove the white cap, and sprinkle the NanoExpo powder on the wound bed. **A thin layer of powder, covering the entire wound bed,** is enough for healing.
3. To empty the glass, gently tap the end of it with your fingers during use.
4. The maximum time to replace the product is **72 hours** and the maximum duration of use is less than 30 days.
5. To remove NanoExpo from the wound bed, use normal saline.
6. If the wound is dry, sprinkle NanoExpo on the wound, then moisten it by sterile water or saline solution. Do not add too much liquid to the dressing to avoid maceration.
7. Moistening NanoExpo with anything other than sterile water or saline solution is not recommended.
8. NanoExpo is as a primary dressing and should be in **direct contact with the wound bed**.

Intended Users (target users)

Healthcare professionals and technicians in the field of wound care and healing

Target population

Adults (excluding pregnant)

Target location (application site in the body)

In direct contact with patients' breached skin

Intended environment and place of use

This product has no restriction or limitation for place of use.

sizes and catalogue numbers:

0.5 g	EP/01/01
1 g	EP/02/01
2 g	EP/03/01

Symbols

	Manufacturer
	Date of manufacture
	Use-by date
	Lot Code
	Catalogue number
	Sterilized using irradiation
	Upper limit of temperature

	Do not resterilize
	Do not re-use
	Consult instructions for use
	Do not use if package is damaged
	Keep dry
	Keep away from sunlight

Manufactured by: Nano Fanavaran Narin Teb

No. 1462, Pharmaceutical Incubator Center of Tehran University of Medical Sciences, North Kargar Street, Tehran, Iran

Please report any adverse events related to this product to the email address of Nanonarin Teb Company (info@nanonarin.com) or to the General Department of Medical Equipment of Iran.

Please contact Nano Narin Teb patient support for further information and directions.

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